

3S50

GB Knee Sleeve Protector

Streifeneder
ortho.production GmbH

Moosfeldstrasse 10
82275 Emmering/Germany
Tel +49 8141 6106-0
Fax +49 8141 6106-29
export@streifeneder.de
www.streifeneder.de

Application

The Knee Sleeve Protector 3S50 is to be applied on lower limb prostheses with sealing sleeves in order to enhance longevity.


Sizes

3S50/S	for ContexGel Knee Sleeve, size S
3S50/M	for ContexGel Knee Sleeve, size M
3S50/L	for ContexGel Knee Sleeve, size L
3S50/XL	for ContexGel Knee Sleeve, size XL and XXL

Description

The Knee Sleeve Protector 3S50 has been generated to enhance the longevity of the Streifeneder ContexGel sleeves. It reduces friction between the sealing sleeve and the socket's trim line and guarantees even for ContexGel sleeves with little leaks a perfect seal.

Instructions

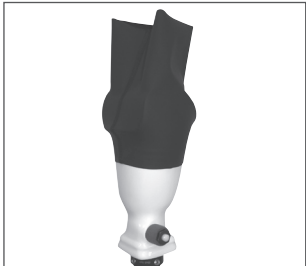

Bevor first use the Knee Sleeve Protector is to be applied on the exterior posterior segment of the prosthetic socket. Make sure, that the narrower part of the protector is about 2,5 cm (1") below the posterior socket trim line. The seam of the protector should also be in the posterior area.

There is no additional tape needed for fixation because of an applied silicone line on the internal fabric which provides enough adhesion to the socket.

The duration of our manufacturers warranty varies by product and service, but all are effective from the date of delivery to the patient. Streifeneder's sole obligation under this manufacturers warranty shall be repair, replace, or refabricate at no charge, or to refund the cost of the system or component thereof to the original purchaser, at Streifeneder's sole discretion. The specific terms and conditions of the manufacturers warranty for the following product are as follows:

Warranty Duration: 3 months

This manufacturers warranty does not cover check sockets, or damage due to accidents, neglect, misuse or operation beyond capacity, parts damaged by improper installation, substitution of parts not approved by Streifeneder ortho.production GmbH, any alteration or repair by others that, in Streifeneder's judgement, materially or adversely affects the product or part. The express warranties set forth hereinabove are in lieu of all other warranties express or implied, including without limitation, any warranties of merchantability or fitness for a particular purpose, and all such other warranties are hereby disclaimed and excluded by Streifeneder ortho.production GmbH. In no event shall Streifeneder's liability of any kind include any special, incidental or consequential damages, even if Streifeneder shall have been advised of the possibility of such potential loss or damage. Some states do not allow the exclusion or limitation of incidental or consequential damages, so the above limitation may not apply for you. This product is destined for exclusive use of the original user and not suitable for re-use.

Declaration of Conformity

Streifeneder ortho.production GmbH declares as manufacturer in sole responsibility, that the Knee Sleeve Protector 3S50 complies with the requirements of directive 93/42/EWG.

Note

The distal area of the ContexGel sleeve must have direct circumferential contact to the shaft! This can be achieved when it overlaps the material of the Knee Sleeve Protector end at least 2,5 cm (1").

Doffing the Knee Sleeve

- Roll the knee sleeve down onto the prosthesis.
- Step out of the prosthesis.

Cleaning Instructions for sealing sleeve

Please wash with unscented mild anti-microbial soap and rinse to remove soap. Dry with a towel. The sleeve can either be completely removed for cleaning or left everted on the socket.

Attention!


Incorrect installation of the Knee Sleeve Protector can result in sleeve failure.

- Please take always care when gathering and rolling the sleeve. Kneeling on hard surfaces as well as excessive pulling or other misuse can tear the sleeve.
- Trimlines should be smooth and low (**NO** supracondylar or supra-patellar socket designs) and should provide smooth transitions from socket to the liner.
- When the sealing sleeve rolls down from the thigh, a larger size might be necessary (thermoforming on a mold can also help).

Manufacturers Warranty

Streifeneder ortho.production GmbH warrants all of its products and services, to the original purchaser, to be free from defects in materials and workmanship. This warranty applies, subject to normal wear and tear, when the products are used as intended, without unapproved modifications, following all instructions and requirements; and when they are fitted by or under the direct supervision of certified/licensed practitioners.

Donning the Knee Sleeve

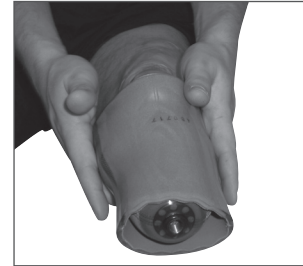


Don the liner.






Don the ContexGel sleeve over the liner and make sure there is enough direct skin contact to the thigh.

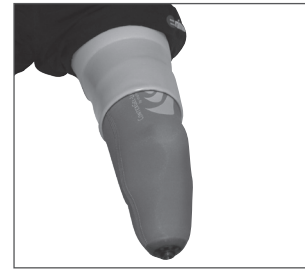

Fold the distal part of the knee sleeve back to the thigh.

When stepping into the prosthesis, make sure the protector does not flap in certain areas into the prosthetic socket. This can cause undesirable peak pressure and can lead to premature wear.

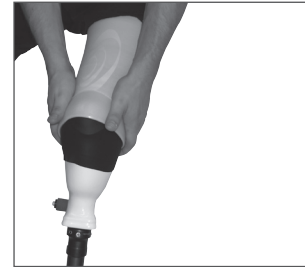

Now roll the sealing sleeve carefully over the Knee Sleeve Protector and the prosthetic socket. The system seals now.

