


ALIGNMENT OF COLLEGE PARK FEET WITH MEDI KNEES

All Medi knees should be set up such that the load line, once the appropriate socket flexion has been incorporated, should normally fall through the centreline of the shin tube.
 Provided a rigid shin tube is being used, the load line can be permitted to fall forward of the centreline by up to 10mm.

With CP feet, if the load line is kept slightly anterior to the centre line, though the feet do differ slightly from one another, they will all function satisfactorily. This is helped by the fact that, in a transfemoral application, CPI suggest the load line should be slightly less anteriorly placed (30/70%), compared with the transtibial application (33/67%).

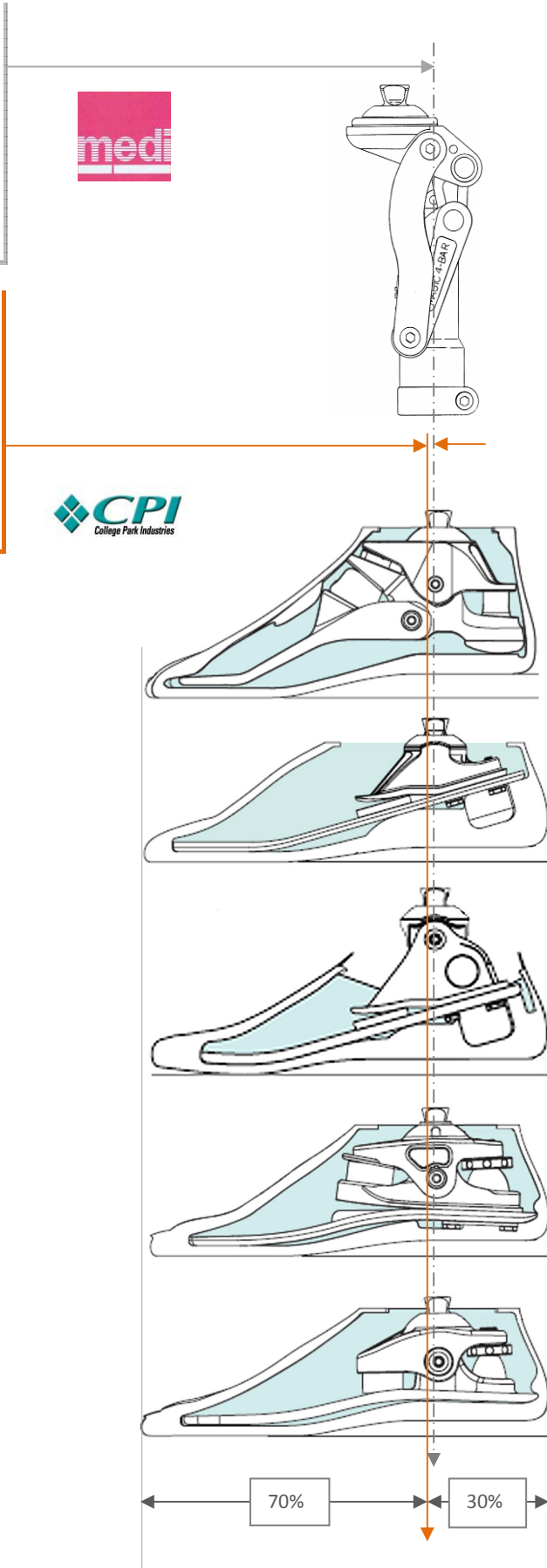

TRUSTEP

TRES

ACCENT

VENTURE

TRIBUTE